



# INTRODUCING **First To Know<sup>®</sup>** **Pregnancy Test**

Featuring Touch-to-Test Technology  
Sample is applied directly to the tip of the test

First To Know<sup>®</sup> is a diagnostic test used for the early detection of the pregnancy hormone beta-hCG in the blood. First To Know<sup>®</sup> is easy to use and gives you a reliable result in the privacy and comfort of your home.

## **EARLY**

When the pregnancy hormone (hCG) enters the bloodstream, First To Know<sup>®</sup> can detect it. The First To Know<sup>®</sup> test can detect pregnancy after the 7th-8th day of conception.

## **SENSITIVE**

First To Know<sup>®</sup> has a high sensitivity of 10mIU/ml, this means that it can detect minimal quantities of the pregnancy hormone beta-hCG.


## **EASY**

Just a small puncture of the finger to obtain a blood drop is necessary to perform the test, after only a 10 minute wait, the result can be read.



# Administering Test \*

1. Perform finger stick.

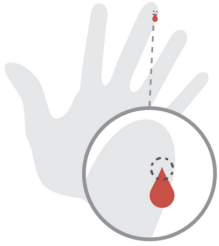

2. Touch the drop of blood to the tip of the test.



3. Set timer for 10 minutes and read result.



## INTERPRETATION OF RESULTS

A positive result has TWO LINES, one at the "C" (Control Line) and one at the "T" (Test Line). The intensity of the lines may vary. Look carefully as lines might be faint.



Positive Result



Negative Result



Invalid Result



Invalid Result

\* For complete instructions, refer to the Product Insert before administering the test.

Visit [NOWDx.com](http://NOWDx.com) for more information.

Description	Quantity	Catalog Number
First To Know® hCG Test Kit	1	8001-1CE
First To Know® hCG Test Kit	2	8001-2CE

